

Click here for our website



Portable Medical Fluid Pump

Remove body fluids easily, safely & quickly

السرّواد

رّواد للخدمات الطبية

ROWAD MEDICAL SERVICES

www.alrowad-ksa.com

Al sulaimaniyah Dist - Riyadh
Saudi Arabia

info@alrowad-ksa.com

mob: +966 58 274 8097

by WK - MedTec GmbH



A conventional syringe is currently used to remove dangerous fluid accumulation in the lungs or abdomen.

Unpleasant for the patient, exhausting and time-consuming for the attending doctor.

**We offer the most efficient method. Easy, safe & quick :
Portable Medical Fluid Pump**

Patented & Certified
Innovation



Development in
close cooperation
with medical
specialists.



Portable Medical Fluid Pump

The innovative method of liquid aspiration

- **Electrically operated with five different speed settings (40–90 ml/min)**
- **Saves time & personnel resources**
- **Portable & single hand operation**
- **More comfortable for patient & doctor**
- **Ideal for ambulance**
- **Single use**
- **Class 1 medical product**

Simply operated, e.g., with a pleural puncture kit.



Conformity assessment:

Article 19 (EU Declaration of Conformity)

Standards applied:

EN 1041:2008+A1:2013

EN 60601-1:2006 + Cor. :2010 + A1:2013

EN 60601-1-2:2015

EN 60601-1-6:2010 + A1:2015

EN ISO 14971:2019

EN ISO 15223-1:2016

MEDDEV 2.7/1 Revision 4



Portable Medical Fluid Pump

Easy connection, e.g. with pleural puncture set

- 1 Remove the fluid pump from the blister package
- 2 Remove protection card
- 3 Slide the switch up (+) and down (-), LED will light
- 4 Properly connect the fluid pump to the intended drainage source (hose with internal diameter of 3.0-4.0 mm)
- 5 Push switch to the suitable level per patient situation
- 6 During operation, the flow rate can be regulated per actual volume of liquid
- 7 The product can be placed flat on the table during sucking

